

INSTRUCTIONS FOR USE

DECONTAMINATION SYSTEMS FOR DL210 AND DC310 DENTAL UNITS

Cleaning the Spittoon Bowl and Aspirators'hoses.

To ensure safety and well-being for everyone in your dental office it's critical to maintain cleanliness and sanitation of waterlines, waste branches and amalgam separators. That's why we have developed special electronics for our dental units to make cleaning and asepsis procedures more simple and effective. There are two cleaning modes to choose from:

Standard cleaning mode. Use this mode to clean the spittoon bowl and aspirators' hoses. Run this mode at the beginning/end of the day or before leaving the dental unit idle for a certain period of time. This is a one-time cycle.

Automatic cleaning mode. Using this mode enables you to maintain cleanliness of aspirators' tubings. Cleaning cycle activates as soon as the aspirator is back in holder. Suction tubings and waste lines are being flushed with the water mixed with disinfectant for 7sec.

The system delivers a required amount of solution into rinsing water automatically.

For Cattani or Metasys separators, it is recommended to use Green&Clean M2 cleaning agent. Use this cleaning agent for *Standard* cleaning (to disinfect bowl flush system and suction tubings) at least twice a day.

It is not obligatory to use *Automatic* cleaning mode.

Use plastic bottle marked with a green "Green&Clean M2" tag as a reservoir for the disinfecting agent.




Caution

Always turn the main switch off before refilling the reservoir with disinfectant.

INSTRUCTIONS FOR USE

Automatic cleaning mode for aspirator's hoses


Use PR1 switch to activate or deactivate the mode. The switch is located inside the spittoon block.

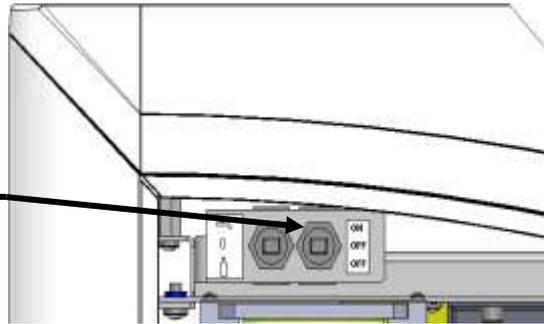

Cleaning cycle starts as soon as both aspirators are back in holders (suction remains active for the next 7s) when this mode is enabled. Suction tubings and waste lines are being flushed with water mixed with disinfectant (Green&CleanM2). Rinsing takes approx. 7s.

Standard cleaning mode for the spittoon bowl and aspirator's hoses

Use T1 button to activate or terminate the *Standard* cleaning cycle

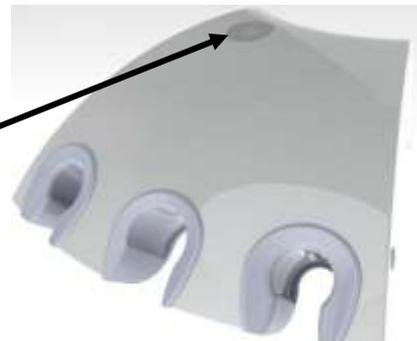

Use the „Standard“ cleaning cycle to rinse the bowl, suction tubings and its waste branches at once. Complete cleaning cycle duration is 55s.

Press the T1 button to start the cycle. You can also use this button to terminate the running cycle. Both aspirators should remain in holders during the whole cycle. The cycle terminates automatically as soon as one of the aspirators is withdrawn from the holder.



Caution

You cannot run any of the two cleaning cycles when the disinfectant amount is low. The lack of disinfectant is indicated by a beeping sound when you try to activate the cycle.