

Annex

SYSTEM OF HYGIENE OF THE DENTAL UNIT DC 350, DL320

1. PURPOSE AND USE OF HYGIENE

Hygiene = Disinfection + Decontamination

„HYGIENE“ is a joint title comprising 2 independent modes (DISINFECTION and DECONTAMINATION).

Disinfecton of *the cooling of instruments and cup filling* is made in order to remove and/or reduce biofilms created by microcolonies of bacteria, moulds and protozoa on the internal surface of water pipes of the dental unit.

Decontamination of *the bowl flushing and suction hoses* is made in order to remove pathogens and to clean the waste and suction system, separator and/or amalgam trap installed in the dental unit.

Control electronics (hardware) of the dental unit is designed to enable the modes of disinfection and decontamination. Disinfection and/or decontamination is *targeted* (it needs to be started manually – it does not work continuously) and it is *one-off* (the mode is not saved in the memory). These 2 independent modes serve for the *hygiene* of the unit at the beginning and/or end of the working day and/or after an extended shutdown of the unit.

Control software of the unit ensures automatic addition of:

- **disinfecting solution (ALPRON)** by firm Alpro medical, GmbH into the cup filling and into the cooling water of instruments,
- **decontamination solution*** into the water for bowl flushing and into the suction hoses.

*In the event that the separator **CATTANI** or amalgam trap **METASYS** is built in the dental unit, it is necessary to use the **GREEN & CLEAN M2** agent for the decontamination. The said agent must be used at least 2 x during every working day for the decontamination of the bowl flushing and suction hoses.

Disinfection of the cooling of the instruments and the cup filling is recommended to perform at least always at the end of every working day. Plastic vessel marked with a **blue label** bearing the title **ALPRON** serves as the store container of the solution for the disinfection of the cooling of the instruments and the cup filling. Plastic vessel marked with a **green label** bearing the title **GREEN & CLEAN M2** serves as the store container of the decontamination solution. The store containers are placed in the spittoon block on the pull-out holder that makes replenishment easier (fig.1).



CAUTION

The modes of disinfection and decontamination are to be activated only if there is sufficient amount of solutions and distilled water in the store containers! Replenishment of the solution into the store containers is possible only if the selector switch of the source of water in position „0“ (Fig. 1), and/or when the main switch of the dental unit is turned off!

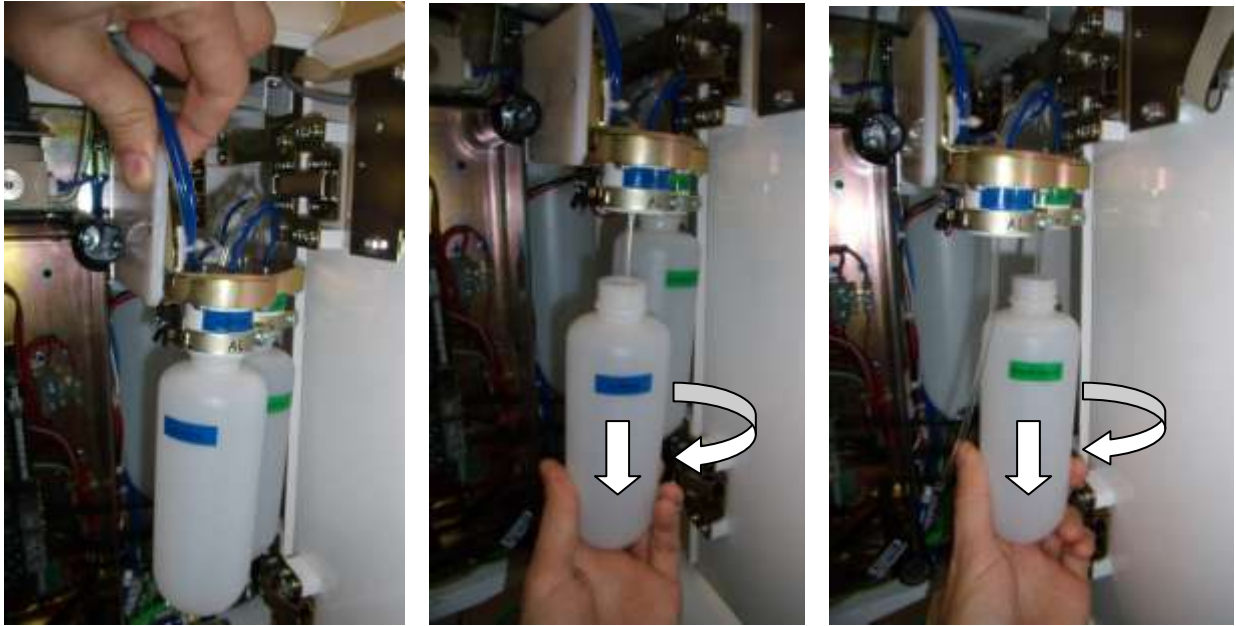


Fig. 1 store containers solutions ALPRON and GREEN & CLEAN M2

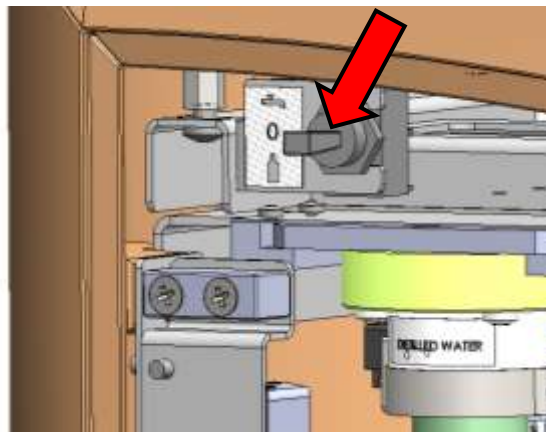


Fig. 2 Selector switch of the source of water in position „0“ (in the spittoon block)

2. CONTROL OF HYGIENE FROM THE KEYBOARD OF THE ASSISTANT TABLE

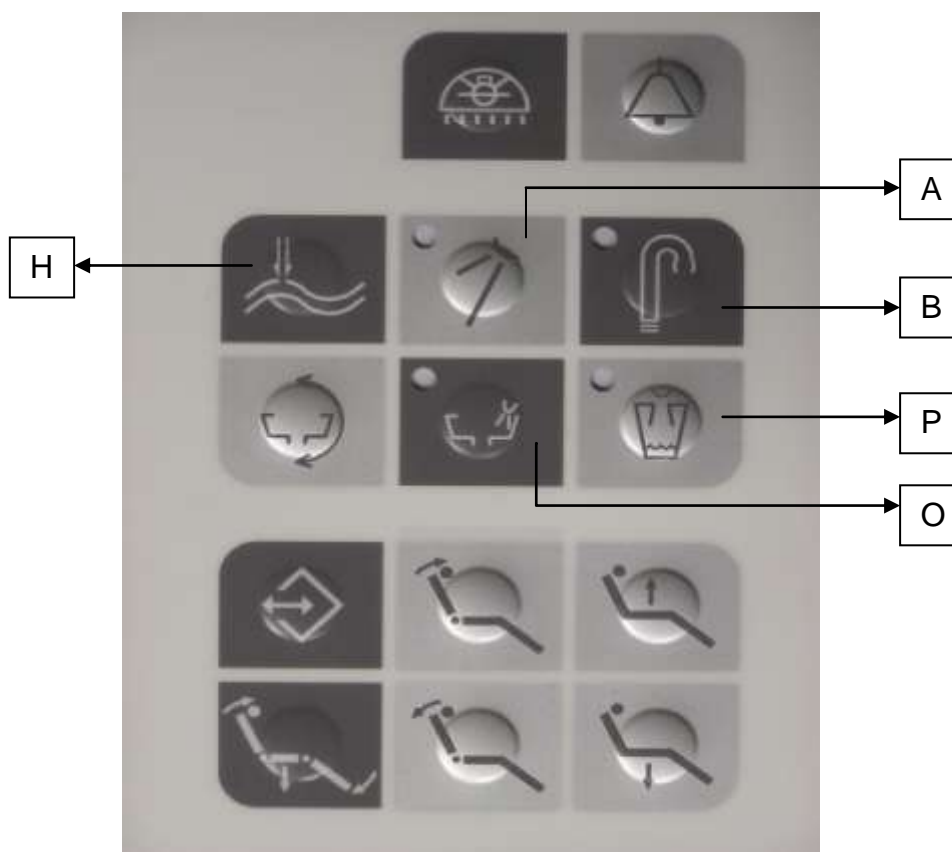


Fig. 3 Keyboard of the assistant table

Legend:

- H** – button of starting the hygiene,
- A** – button of the disinfection of the cooling of instruments,
- B** – button of the decontamination of suction hoses,
- O** – button of the decontamination of the bowl flushing,
- P** – button of the disinfection of the cup filling.

3. CONTROL OF HYGIENE FROM THE TOUCHSCREEN OF THE ASSISTANT TABLE

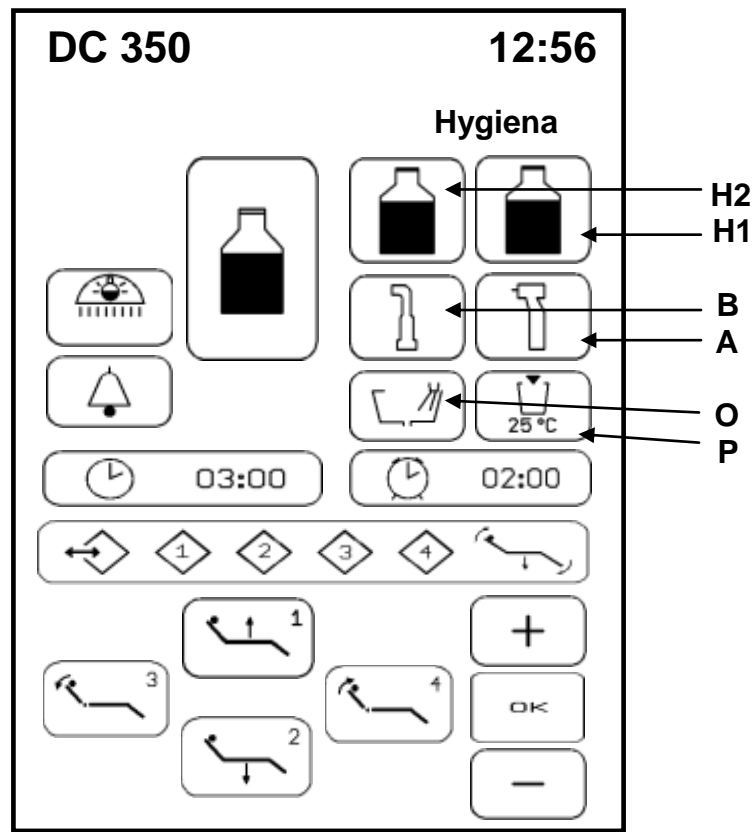


Fig. 4 Touch screen of the assistant table

Legend:

- H1** – button of starting the disinfection + indication of the lack of the solution for disinfection,
- H2** – button of starting the decontamination + indication of the lack of the solution for decontamination,
- A** – button of the disinfection of the cooling of instruments,
- B** – button of the decontamination of suction hoses,
- O** – button of the decontamination of the bowl flushing,
- P** – button of the disinfection of the cup filling.

4. INDICATION PANEL OF THE SPITTOON BLOCK

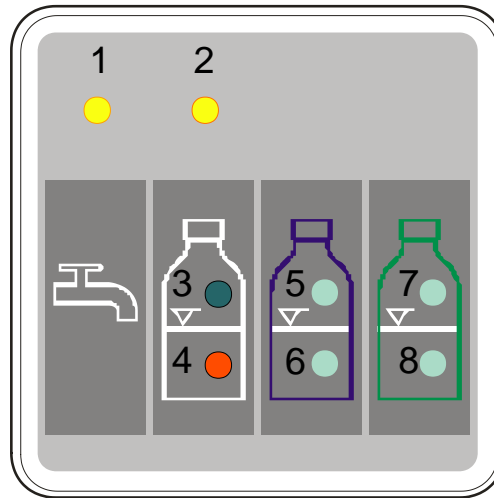


Fig. 5 Indication panel of the spittoon block

Legend:

- 1 - **yellow led**, supply of water from the central distribution,
- 2 - **yellow led**, supply of distilled water from the bottle,
- 3 - **green led**, indicates there is enough distilled water in the bottle,
- 4 - **red led**, indicates there is a lack of distilled water in the bottle,
- 5 - **green led**, indicates there is enough solution for disinfection,
- 6 - **red led**, indicates there is a lack of solution for disinfection,
- 7 - **green led**, indicates there is enough solution for decontamination,
- 8 - **red led**, indicates there is a lack of solution for decontamination.

Lack of distilled water is indicated by flashing the red led diode and/or audible signal. Sufficient amount/ lack of the solutions of hygiene is indicated also on the touchscreen.

5. ACTIVATION AND INDICATION OF HYGIENE

Hygiene can be activated by means of the keys of the keyboard (Fig. 4) or the touchscreen (Fig. 5) of the assistant table.

5.1 Description of the functions of the keyboard

For simplification the description is concerned only with the hygiene part, the buttons are unambiguous (Fig.4).

5.1.1 Decontamination of the bowl flushing

- **Activation:** program the bowl flushing time to **min. 10 seconds**.

Press the button **H**: , then the button **O**: .

Thus the flushing of the spittoon bowl with water with added decontamination solution will take place for pre-programmed time. After that the process will stop itself. Decontamination is one-off, it means that the mode is not saved in the memory.

- **Cancellation:** decontamination can be cancelled by repeated pressing of the flushing button **O**, double pressing of the button **H** or by turning the unit off.
- **Indication:** decontamination is indicated by the **blue led diode** on the button **O**.

5.1.2 Decontamination of suction hoses


- **Activation:** press the button **H**: , then the button **B**: .

Both aspirators situated in the holders will be flushed with water with added decontamination solution for **60 seconds**. Decontamination is one-off, it means that the mode is not saved in the memory.

- **Cancellation:** decontamination can be cancelled by double pressing of the button **H**, taking of any suction hose out of the holder and/or by turning the unit off.
- **Indication:** decontamination is indicated by the **blue led diode** on the button **B**.

5.1.3 Disinfection of the cup filling

Fit the drainage funnel (placed in the pack furnished to the dental unit) over the special container (Fig. 6a) into the Ø16 mm opening. Fit the special container, prepared as described above, over the spittoon bowl (Fig. 6b – cup filling must be directed into the drainage funnel).

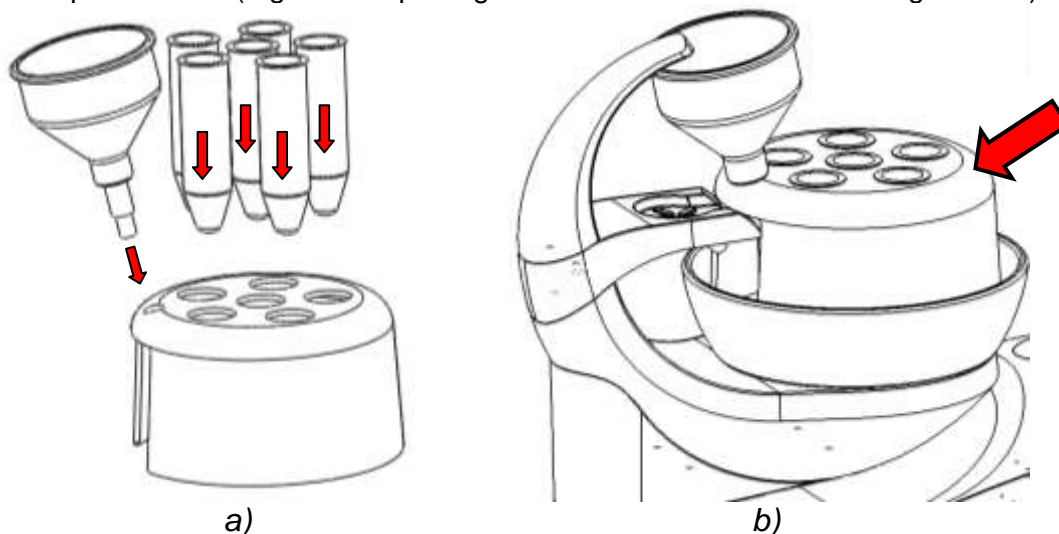

Obr. 6 Container for disinfection the instruments and cup filling

Program the cup filling time to **min. 5seconds**.

- **Activation:**

Press the button **H**:  , then the button **P**:  .

Thus the cup filling is started with water with added disinfecting solution for pre-programmed time. Disinfection is one-off, it means that the mode is not saved in the memory.

- **Cancellation:** disinfection can be cancelled by repeated pressing of the button **P**, double pressing of the button **H** or by turning the unit off.
- **Indication:** Disinfection is indicated by the **blue led diode** on the button **P**.

It is recommended to perform disinfection at the end of each working day. Subsequent rinsing of the cup filling with clean water (by pressing the button **P**) should be made for **min. 20 seconds**. The said rinsing is recommended to be made only at the beginning of the next working day.

5.1.4 Disinfection of the cooling of instruments

Insert the beds for the instruments (Fig. 6a) into 6 openings of Ø36 mm in the special container. Fit the special container, prepared as described above, over the spittoon bowl (Fig. 6b). Take the instruments on which you want to apply disinfection. Place them into the beds in the special container on the spittoon bowl (Fig. 7).



Fig. 7 Container for the disinfection of Instruments

- **Activation:**

Press the button **H**:  , then the button **A**:  .

Thus the filling of the cooling distributions of the instruments with disinfecting solution will start automatically. **Filling** with the solution takes **30 seconds** (a message is displayed - Fig. 8a). Then all the valves are closed and the **solution acts** for the **set time** (a message is displayed – (Fig. 8b). After the time has elapsed, all the valves of water of the instruments are opened again and **rinsing** will take place for **20 seconds** (a message is displayed - Fig. 8c).

- To cancel the disinfection: press the button **A** again, press the button **H** twice and/or turn off the unit.
- Indication: disinfection is indicated by means of a blue LED diode on the button **A**.

5.1.5 Disinfection of the syringe

It serves to perform the hygiene of the syringe fitted on the control panel of the dentist and the syringe fitted on the assistant table (if there is any).

CAUTION

It is possible to start the disinfection of the syringe only when at least one instrument, on which the disinfection is to be performed too, is taken out of the holder on the control panel. For the reason stated, it is necessary to carry out the disinfection of the syringe simultaneously with the disinfection of other instruments (Clause 5.1.4).

- **Activation:**

Make sure that at least one instrument is taken out of the holder on the control panel, besides the syringe, and that it is placed in the special container (Fig. 7) in order to perform disinfection. Take the syringe out of the holder of instruments and direct it toward the spittoon bowl and/or into the free opening in the special container, into the drainage funnel (Fig. 7) or into other external vessel.

Press the button **H**:  , then the button **A**: .

Thereafter press the left lever on the syringe (water), keep the syringe still directed into the spittoon bowl. **Keep** the left lever pressed for **ca 20 sec**. **Then release** the lever and **replace** the syringe **in the holder** on the control panel and/or on the assistant table. Thus filling the water distribution of the syringe with disinfecting solution. Disinfection of other instruments continues in automatic mode described in Cl. 5.1.4.

- **To cancel the disinfection:** press the button **A** again, press the button **H** twice and/or turn off the unit.
- **Indication:** disinfection is indicated by means of a **blue LED diode** on the button **A**.



It is recommended to perform the disinfection of syringes at the end of each working day. Subsequent rinsing of the syringes with clean water should be made for the period of **at least 20 seconds** (direct the syringe into the spittoon bowl, press the left lever). The said rinsing is recommended to be made at the beginning of the next working day.

5.2 Description of the functions of the touch screen on the assistant table



Control elements on the touch screen (Fig. 4) of the assistant table are similar to those on the keyboard. Six representations of the keys of hygiene is divided into two groups – trios one under another. Symbols of bottles serve to indicate the level and in the same time to start the process of hygiene. Press the symbol of the respective bottle and then the symbol of the part of the distributions in which the hygiene is to take place. **All the conditions and processes are identical with that for the control from the keyboard (Clause 5.1).**

5.2.1 Decontamination of the bowl flushing

Program the bowl flushing time to **min. 10 seconds**.

Press the button **H2**:  , then the button **O**: .

5.2.2 Decontamination of the suction hoses

Press the button **H2**:  , then the button **B**: .

5.2.3 Disinfection of the cup filling

Place a cup with the volume of min. 100 ml under the tube of the cup filling. Program the cup filling time to **min. 10 seconds**.

Press the button **H1**:  , then the button **P**: 

5.2.4 Disinfection of the cooling of the Instruments

Press the button **H1**:  , then the button **A**: 

5.3 Description of the functions and representations of the touchscreen on the control panel of the dentist

5.3.1 Setting of the treatment time of the solution for the disinfection of the cooling of the instruments

Setting of the treatment time of the solution is made on the touchscreen of the control panel of the dentist (Fig. 8, 9).

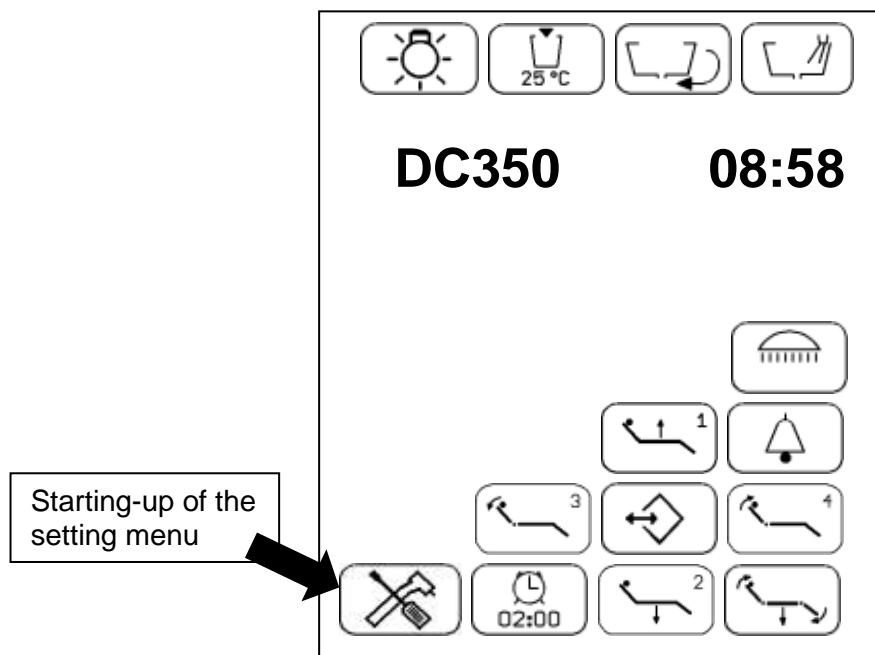


Fig. 8 Main page (touchscreen of the control panel of the dentist)


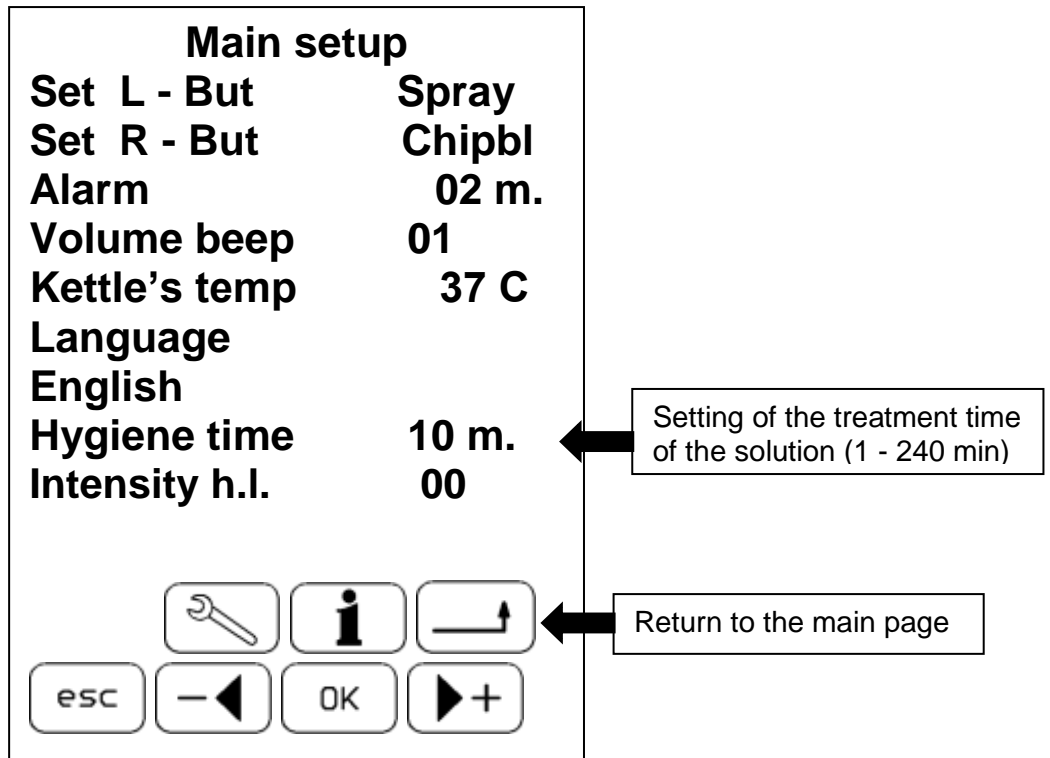


Fig. 9 Setting of the treatment time of the solution

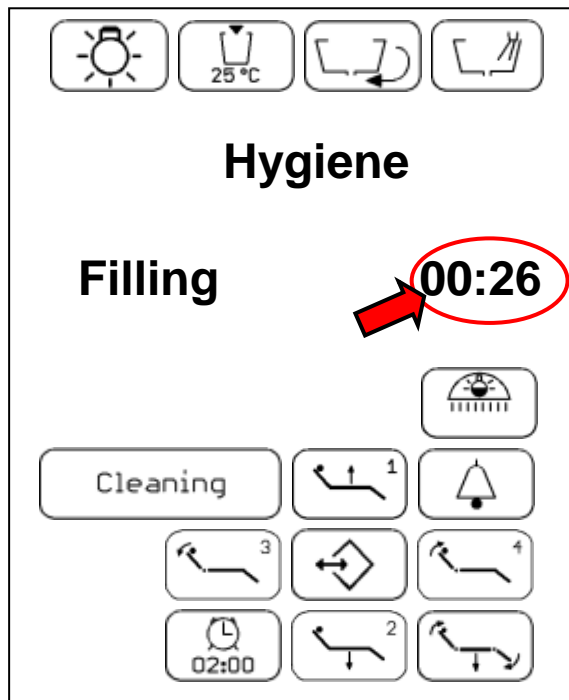

a) filling with the solution (30 s)


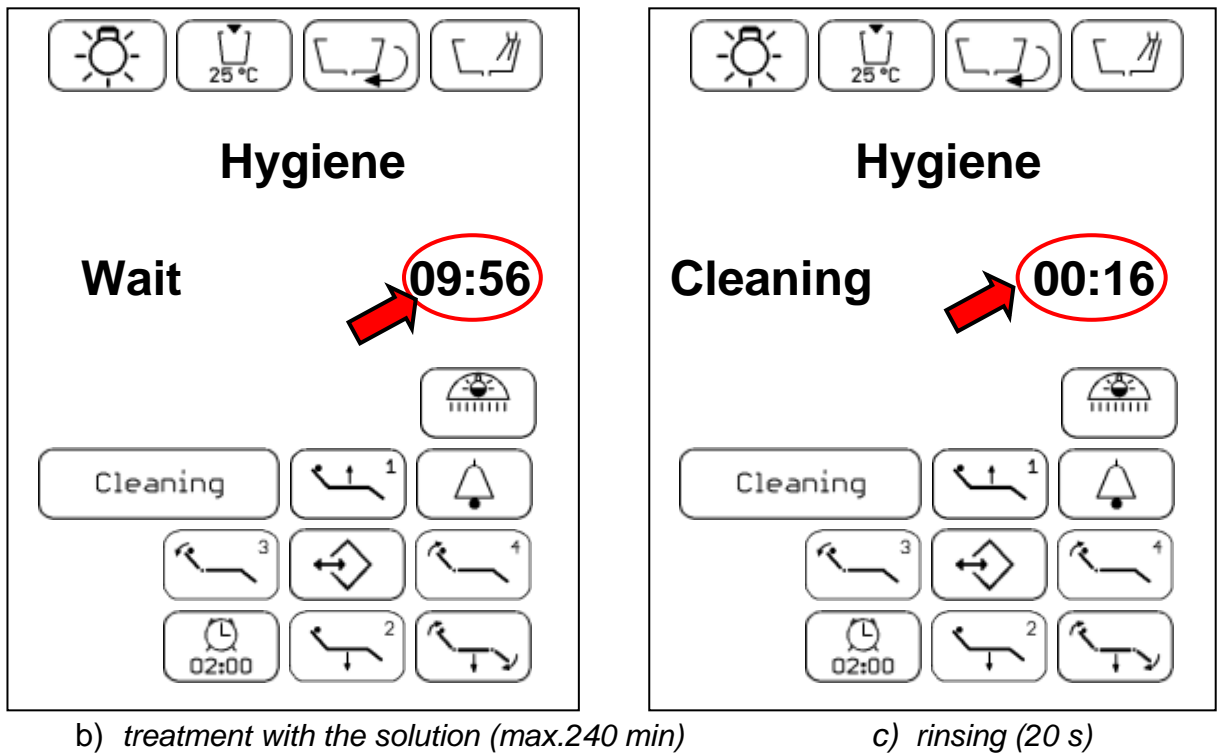


Fig. 10 Displays of the touchscreen during the hygiene of the cooling of the Instruments.

5.3.2 Interruption of the disinfection of the cooling of the instruments

If during the hygiene of the instruments the unit is turned off or the instruments taken or replaced, a message is displayed on the touchscreen of the dentist (Fig. 11). In this case hygiene regime end rinses press **Cleaning**.

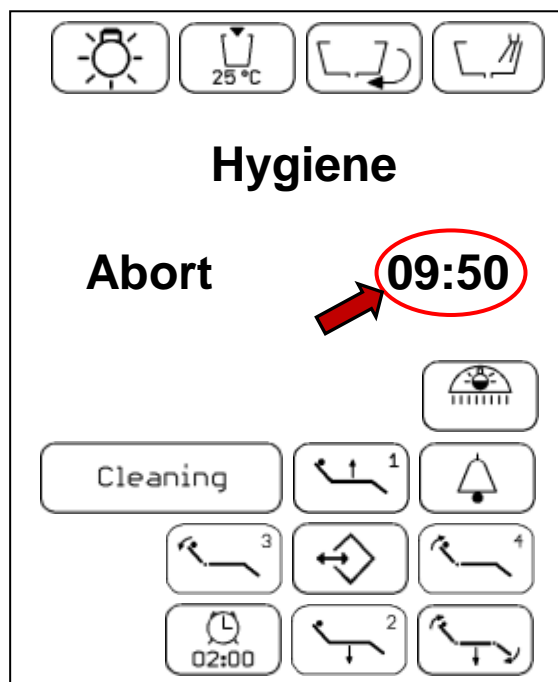

Obr. 11 Display of the time elapsed at the interruption of the disinfection of the cooling of the instruments

Instruments the taking of which was changed are displayed on the touchscreen of the dentist (Fig. 12).

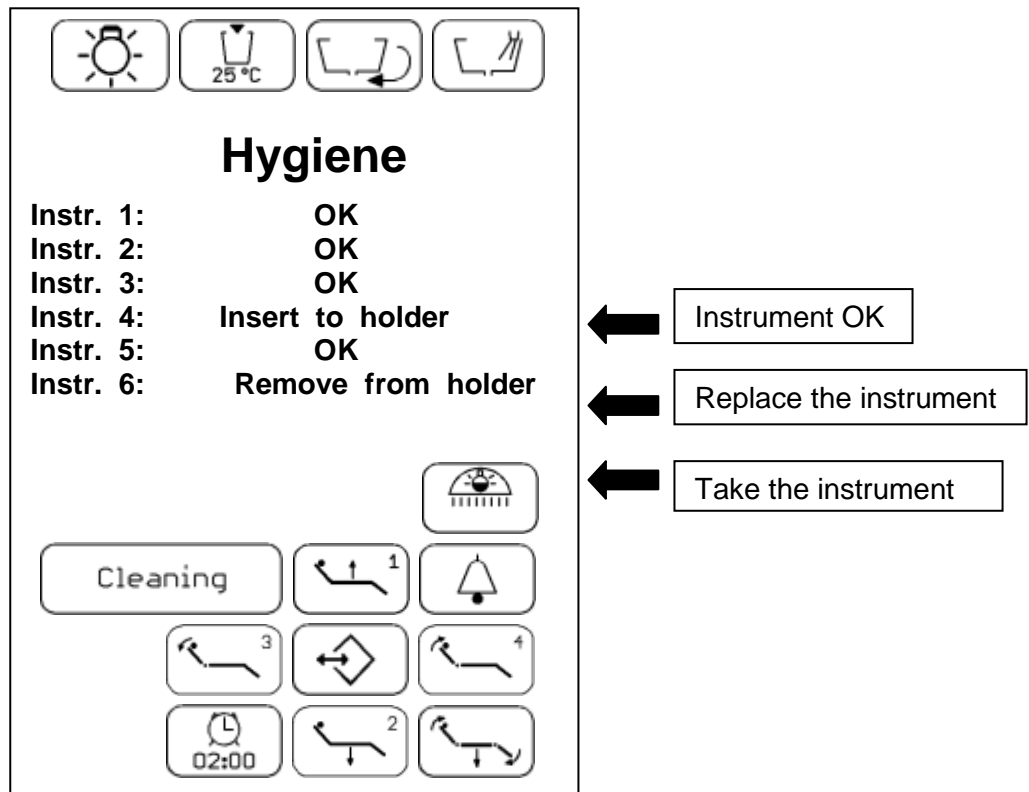


Fig. 12 Display of the taken instruments at the interruption of the disinfection



CAUTION

The control panel will not enable any other activity until the rinsing has taken place with the same taken instruments with which the process was started!

6 RECOMMENDATION OF THE MANUFACTURER OF THE DENTAL UNITS

In general the targeted hygiene of the dental unit is the most efficient **at the end of the working day**. Subsequent rinsing of the internal distributions of the unit **at the beginning of the next working day** with clean and/or distilled water will ensure washing off of the disrupted biofilms (after treatment with disinfecting and decontamination solution) from the internal distributions of the dental unit.